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ART. 6040

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KNEE JOINT SUPPORT ELASTIC

Indications for use:

Can be used in cases of knee joint traumas, sprain of ligaments, arthritis, arthrosis, during rehabilitation after plaster cast. Also used for prevention of common dislocations and traumas while physical exercises.

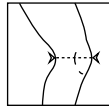
Application:

- Provides soft and reliable fixation and compression of knee joint
- Breathable fabric stretches in four directions for comfortable use
- Fits for left and right leg

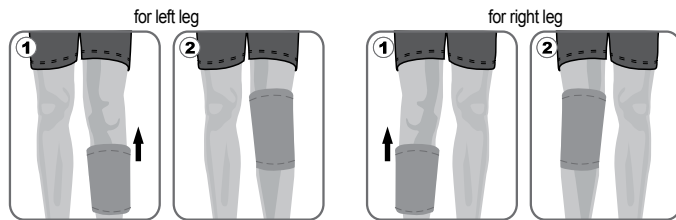
Contraindications:

Inflammatory skin disease in the area of brace application, individual intolerance to the brace materials.

Dimensions	Sizes				Max deviation
	S	M	L	XL	
Knee circumference, cm	28-33	34-39	40-45	46-51	± 1,0 cm

How to get it on:

Maintenance:

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

Precautionary measures:

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

Contents:

nylon - 35%, latex - 30%, cotton - 25%, polyether - 10%.

Storage conditions:

store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

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