KZ

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INSTRUCTION MANUAL

LH Urine Ovulation Test (Test Strip)

Home Test LH Urine Ovulation Test is a test that helps to predict when there is a LH (luteinizing hormone) surge, and in turn, when you are likely to ovulate. During the ovulation the egg is released from the ovary. As the female egg can be fertilized only during 6-24 hours after ovulation early detection of ovulation is very important.

When to begin testing

First you must determine the length of your menstrual cycle. This is the number of days from the first day of your menstrual bleeding to the day before your next bleeding begins again. Please refer to the chart to determine when you should start testing. If your cycle is shorter than 21 days, or longer than 38 days, consult your doctor. If you do not know your cycle length, you may begin the test 11 days after your first period since the average cycle length is 28 days. Perform 1 test each day over a 5 day period or until the LH surge has been detected.

Urine collection

1. Do not use first morning urine samples as LH is synthesized in your body early in the morning. It will be shown up in your urine later in the day.
2. The best time to collect urine is between 10:00 am and 08:00 pm.
3. Collect urine at about the same time each day.
4. Reduce liquid intake about 2 hours before collecting your urine as a diluted urine sample can prevent the test from detecting LH surge.

Test procedure

1. Test should be performed at room temperature (from 2°C to 30°C).
2. Remove the test strip from the sealed pouch.
3. Holding the test strip vertically immerse it into urine with the arrow end pointing towards the urine. Do not immerse the test strip below MAX (maximum) line. Take the strip out after 5 seconds and lay it flat on a clean, dry, non-absorbent surface (e.g. mouth of the urine container).
4. Wait for colored bands to appear. Test results may be observed not earlier than 10 minutes, but not later than 30 minutes.

Your Cycle Length	Start To Test On
21 days	Day 6
22 days	Day 6
23 days	Day 7
24 days	Day 7
25 days	Day 8
26 days	Day 9
27 days	Day 10
28 days	Day 11
29 days	Day 12
30 days	Day 13
31 days	Day 14
32 days	Day 15
33 days	Day 16
34 days	Day 17
35 days	Day 18
36 days	Day 19
37 days	Day 20
38 days	Day 21

Interpretation of results

No LH Surge:

If only one color band appears on the control region or the test band is lighter than the control band there is no LH surge.

LH Surge:

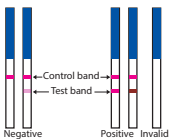
If two color bands are visible and the test band is equal to or darker than the control band it means the ovulation will probably occur in the next 24-48 hours. If you want to become pregnant the best time to have intercourse is after 24 hours but before 48 hours.

Invalid:

No visible bands at all. Repeat the test with a new test strip.

Storage conditions:

The test kit should be stored at room temperature (from 2°C to 30°C) in a sealed foil pouch to the date of expiration. Check the expiration date before use. The test kit should be kept away from direct sunlight, moisture, and heat. After usage throw the test strip in a waste bin.

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