AUTOMATIC BLOOD PRESSURE MONITOR

MODEL M-100A
USER'S MANUAL

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DEAR DR. FREI AUTOMATIC BLOOD PRESSURE MONITOR OWNER

Thank you for choosing Automatic Blood Pressure Monitor **Dr. Frei**® TM model M-100A. We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of Swiss Trademark **Dr. Frei**®.

Before starting to use this device please study the user's manual carefully. The user's manual offers all information you need to correctly measure your blood pressure and pulse correctly. For all questions concerning the device please contact your local distributor or **Dr. Frei**® service center in your country.

ATTENTION

This blood pressure monitor is designed to carry out self-control over blood pressure but NOT to make self-diagnosis of hypertension/hypotension. Please DO NOT diagnose by yourselves basing on the measurement results obtained with the blood pressure monitor. Please DO NOT execute self-treatment of high/low blood pressure and DO NOT change the methods prescribed without consulting your doctor.

Type BF applied part.

Read the instructions carefully before using this device.


TABLE OF CONTENTS

| | |
|--|-----------|
| IMPORTANT INFORMATION ON BLOOD PRESSURE AND ITS MEASUREMENT | 6 |
| What Is Blood Pressure? | 6 |
| Which Values Are Normal? | 7 |
| ADVANTAGES OF AUTOMATIC BLOOD PRESSURE MONITOR MODEL M-100A | 8 |
| Blood Pressure Level Indicator | 8 |
| Irregular Heartbeat Detector | 8 |
| GETTING READY FOR MEASUREMENT | 9 |
| Safety Precautions | 9 |
| Components of Your Blood Pressure Monitor Kit | 9 |
| Description of the Blood Pressure Monitor | 10 |
| Inserting the Batteries | 10 |
| Using an A/C Power Adapter | 11 |
| Cuff Connection | 11 |
| MEASUREMENT PROCEDURE | 11 |
| Fitting the Cuff | 12 |
| Measuring Procedure | 12 |
| Reading Measurement Result | 13 |
| Discontinuing a Measurement | 13 |
| MEMORY FUNCTION | 13 |
| ERROR MESSAGES/TROUBLESHOOTING | 14 |
| CARE AND MAINTENANCE | 15 |
| SYMBOL DESCRIPTION | 16 |
| TECHNICAL SPECIFICATIONS | 17 |
| WARRANTY | 17 |

IMPORTANT INFORMATION ON BLOOD PRESSURE AND ITS MEASUREMENT


What Is Blood Pressure?

Blood pressure is the pressure that your blood exerts to the vascular walls. Blood pressure is necessary to provide for constant blood flow inside the body. Thanks to it the cells get oxygen that provides for their normal functioning. The heart performs the function of a «pump», sending blood to the blood vessels. Each heart beat creates a certain level of the blood pressure.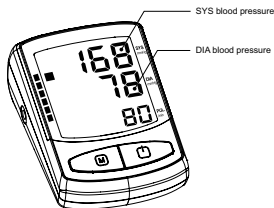

There are 2 kinds of blood pressure: a systolic (upper) one, which corresponds to the heartbeat pushing blood into the arteries; and a diastolic (lower) one, which means the blood pressure between two heartbeats. Blood pressure is subject to fluctuations during the day even in healthy people. The fluctuations are influenced by a number of factors - time of day, person's condition, physical or mental activity, environment, etc.

Day-Night Fluctuations of the Blood Pressure

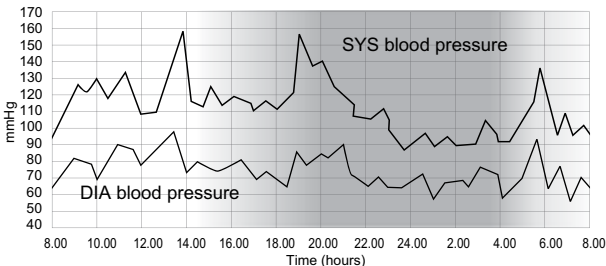An increase of blood pressure increases the burden onto the heart, affects

blood vessels making their walls thick and less elastic.

One of the features of the hypertension is its ability to remain unnoticed for the patient at its early stages. That's why the self-control of the blood pressure is so important. With the illness progressing, headaches and regular dizziness appear, the sight declines, the functioning of vitals (encephalon, heart, kidneys, blood vessels) breaks down. Without special treatment the complications of hypertension might be kidney damages, breast-pang, paralytic stroke, aphasia, dementia, heart attack and stroke.

Which Values are Normal?

The world standard as for the norms of the blood pressure is the Classification* of the World Health Organization (WHO):

| Blood Pressure Category | Systolic Blood Pressure (mmHg) | Diastolic Blood Pressure (mmHg) |
|---------------------------------|--------------------------------|---------------------------------|
| Blood pressure too low | < 100 | < 60 |
| Blood pressure optimum | 100 - 119 | 60 - 79 |
| Blood pressure normal | 120 - 129 | 80 - 84 |
| Blood pressure slightly high | 130 - 139 | 85 - 89 |
| Blood pressure too high | 140 - 159 | 90 - 99 |
| Blood pressure far too high | 160 - 179 | 100 - 109 |
| Blood pressure dangerously high | ≥ 180 | ≥ 110 |

* printed with curtailments

- The diagnosis of hypertension requires from the patient to combine medical treatment prescribed by the doctor and mode of life correction.
- People with normal pressure and high normal pressure are recommended to carry out self-control of their tension in order to timely take measures to decrease the blood pressure level down to the optimal one without using any medications.
- For people more than 50 years old high level of systolic blood pressure (higher than 140 mmHg) is more crucial than diastolic pressure.
- Even with blood pressure being normal, people run the bigger risk of hypertension development with advancing age.

ATTENTION

If you have normal results of blood pressure measured under calm conditions but your results are excessively high when measured under the conditions of physical or mental exhaustion, this might be a sign of so called brittle (that is unstable) hypertension. If you suspect that, please consult your doctor.

When measured correctly, if diastolic blood pressure is more than 120 mmHg, it is necessary to call the doctor immediately.


ADVANTAGES OF AUTOMATIC BLOOD PRESSURE MONITOR MODEL M-100A

Blood Pressure Level Indicator



Blood pressure level indicator is located along the left side in the display. The classification corresponds to WHO (World Health Organization) ranges described in the table of the section «Which values are normal?». After the measurement there appears the color bars in the left part of the display: green zone - optimal blood pressure, yellow - elevated, orange - too high, red - dangerously high. This function helps you to self-orient in the measurement results.

Irregular Heartbeat Detector

This technology indicates allorhythmic heartbeating. If the symbol of IHB  appears on the display that means that certain abnormality in heart beat frequency was detected during the measurement.

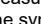In this case, the result may deviate from your normal blood pressure – repeat the measurement. In most cases, this is no cause for concern. However, if the symbol  appears on a regular basis (e.g. several times a week with measurements taken daily) we advise you to tell your doctor. Please show your doctor the following explanation:

Information for the doctor on frequent appearance of the Irregular Heartbeat Detector.

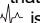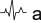This instrument is an oscillometric blood pressure monitor that also analyses pulse frequency during measurement. The symbol  is displayed after the measurement, if pulse irregularities occur during measurement. If the symbol  appears more frequently (e.g. several times per week on measurements performed daily) we recommend the patient to seek medical

advice. The instrument does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

GETTING READY FOR MEASUREMENT

Safety Precautions

1. Use this device **ONLY** for the intended use described in this manual.
2. Do **NOT** use accessories which are not specified by the manufacturer.
3. Do **NOT** use the device if it is not working properly or damaged.
4. Do **NOT** use the equipment where aerosol sprays are being used, or where oxygen is being administered.
5. Do **NOT** use under any circumstances on newborns.
6. This device does **NOT** serve as a cure for any symptoms or diseases. The data measured are for reference only. Always consult your doctor to have the results interpreted.
7. Keep the equipment and its flexible cord away from hot surfaces.
8. Do **NOT** apply the cuff to areas other than the place directed.
9. Proper maintenance and periodically calibration are essential to the longevity of your device. If you are concerned about your accuracy of measurement, please contact local customer service for help.
10. This system is intended for individuals age 18 or above.
11. Do not use it for diagnosis of hypertension or testing on newborns, babies, young children or persons who cannot express their consent.
12. If you have been diagnosed with a severe arrhythmia or irregular heartbeat, atrial or ventricular premature beats or atrial fibrillation, measurements made with this instrument should only be evaluated after consultation with the doctor or your healthcare professionals.


KEEP THIS MANUAL DURING THE WHOLE LIFETIME OF THE DEVICE

Components of Your Blood Pressure Monitor Kit


The blood pressure monitor kit includes Automatic Blood Pressure Monitor model M-100A, M size cuff (22-32 cm), 4 batteries (AA), AC adaptor, user's manual, warranty card, kit box.

Description of the Blood Pressure Monitor

Measuring Unit Model M-100A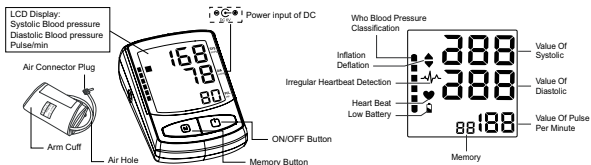


Accessories:

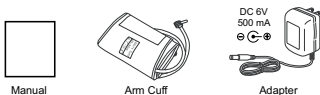* Please use the original adapter in accessory, otherwise it is impossible to achieve the claimed performance.

Inserting the Batteries

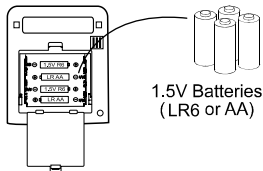1. Remove the battery cover.
2. Insert new batteries into the battery compartment as shown, taking care that the polarities (+) and (-) are correct.
3. Close the battery cover. Use only LR6, AA batteries.


CAUTION


- Insert the batteries as shown in the battery compartment.
- When  blinks in the display, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction.
-  mark does not appear when the batteries run out.
- Battery life varies with the ambient temperature and may be shorter at low temperatures.
- Remove the batteries if the device is not to be used for a long time.
- The batteries may leak and cause a malfunction.
- Use the specified batteries only. The batteries provided with the device are for testing monitor performance and may have a shorter life.

NOTE: To prolong the batteries life the device switches off automatically if no button is pressed for 1 minute. Otherwise you can switch it off by pressing the ON/OFF button.

Using an A/C Power Adapter

It is possible to operate Automatic Blood Pressure Monitor Model M-100A with an AC/DC adaptor.

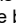1. Push the plug into the socket at the bottom part of the device. No power is taken from the batteries while the AC/DC adaptor is connected to the device.
2. Plug the AC adaptor into a 220/110 V power socket. The device is ready for measurement.
3. Test that power is available by pressing the «  » button.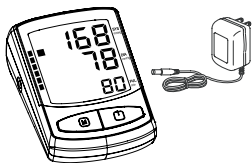


NOTE:

- Please use the AC adaptor of Trademark **Dr. Frei®**. Any other adaptor being used, the service center is not considered responsible for the warranty servicing/repair of the monitor.
- No power is taken from the batteries while the AC/DC adaptor is connected to the device.
- Please consult service center if you have questions relating to the AC/DC adaptor.

Cuff Connection

Insert the cuff connector into the opening provided on the side of the device as shown in the picture.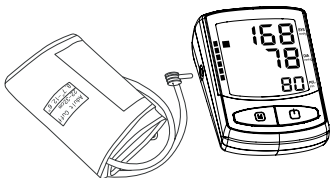

MEASUREMENT PROCEDURE



ATTENTION

- Find time to relax by sitting in a quiet atmosphere for some time before measurement.
- Efforts by the patient to support the arm can increase the blood pressure. Make sure you are in a comfortable, relaxed position and do not flex the arm during measurement.
- Always measure on the same arm (normally left).
- Remove any garment that fits closely to your upper arm. Do not roll the

- sleeve since it can squeeze your hand and this can lead to false results.
- Use only clinically approved original cuff.
- If you want to follow the results of your blood pressure measurements, always perform measurements at the same time of day, since blood pressure changes during the course of the day.
- Measurements should be done after a 5 minute rest to ensure accuracy.

Fitting the Cuff

1. Plug the cuff connector to the cuff socket on the device left side.
2. Stretch your left (right) arm in front of you with your palm facing up. Slide and place the cuff onto your arm to let the air tube and artery mark region toward the lower arm. Wrap and tighten the cuff above your elbow. The artery mark on the edge of the cuff should be approx. 0.8" to 1.2" (2 cm to 3 cm) above your elbow. Align the tube over the main arteries on the inside.
3. Leave a little free space between the arm and the cuff: you should be able to fit two fingers between them. Remove all clothing covering or constricting the measurement arm.
4. Press the hook material firmly against the pile material. The top and bottom edges of the cuff should be tightened evenly around your upper arm.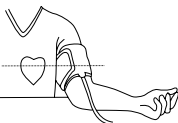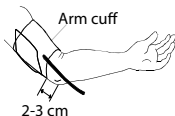


Measuring Procedure

Always apply the pressure cuff before turning on the monitor.


1. Press the «» button. All icons appear two seconds on DISPLAY, then switch to measurement, and display «0» or last measurement record.

2. Pump begins to inflate the cuff. The heart symbol «♥» is shown on the display. When complete, the results will be displayed.
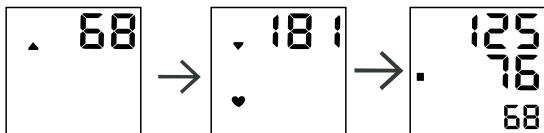


Reading Measurement Results

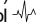When the measurement has been concluded, the measured blood pressure, pulse rate and color bars classified by WHO (World Health Organization) are now displayed. The appearance of this symbol  signifies that an irregular heartbeat was detected.

NOTE: We recommend contacting your physician if you see this indicator frequently. This indicator is only a caution. It is important that you be relaxed, remain still and do not talk during measurements.

On the graphic indicator the element showing level of arterial pressure on a scale of World Health Organization is displayed.

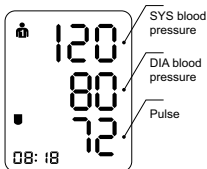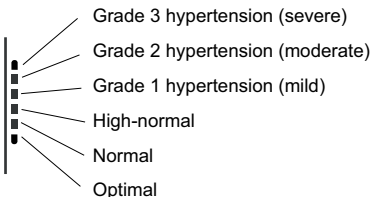Discontinuing a Measurement

If it is necessary to interrupt a blood pressure measurement for any reason (e.g the patient feels unwell), the ON/OFF button be pressed at any time. The device then immediately lowers the cuff pressure automatically.

MEMORY FUNCTION

Automatic Blood Pressure Monitor Dr. Frei TM model M-100A automatically stores the result measured.

To see the result of the last measurement press the «M» button.

If an irregular heartbeat interval was detected in the measurement stored in memory, the icon  will be displayed.

ERROR MESSAGES/TROUBLESHOOTING

If you have trouble in using the unit please check the following points first.

| ERROR DISPLAY | POSSIBLE CAUSE | HOW TO CORRECT |
|--|--|---|
| Nothing is displayed when you push the ON/OFF button or  battery icon flashes | No battery installation | Insert batteries |
| | Battery worn out | Replace with new batteries |
| | The polarities of batteries placed wrongly | Insert battery in the correct polarities |
| E1: Can't normally increase pressure | Check your cuff for air leakage | Replace cuff with new one |
| E3: Inflate pressure too high | | Remeasure or send back to dealer to recalibrate |
| E2 E4: Hand shaking while measurement | Hand or body shaking while measurement | Keep static position and measure again |
| Battery icon on | Battery low power | Replace battery and measure again |
| The systolic pressure value or diastolic pressure value too high | The cuff was held lower than your heart | Keep correct position and measure again |
| | The cuff was not attached properly | |
| | You moved or spoke during measurement | |
| The systolic pressure value or diastolic pressure value too low | The cuff was held higher than your heart | Keep correct position and measure again |
| | You moved or spoke during measurement | |


ATTENTION

Should any technical malfunctions arise in the blood pressure monitor, please contact the authorized service center of **Dr. Frei®**. **Never attempt**

to repair the instrument yourself! Any unauthorized opening of the instrument invalidates all warranty claims.

NOTE: Blood pressure is subject to fluctuations even in healthy people. Please remember that comparable blood pressure measurements always require the same time and the same conditions! These are normally quiet conditions. If you follow the correct procedures described above and still get the fluctuations of blood pressure of more than 15 mmHg and/or you repeatedly hear irregular pulse tones please consult your doctor.

CARE AND MAINTENANCE
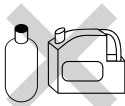
| | |
|---|---|
| <p>Do not expose the device to either extreme temperatures, humidity, dust or direct sunlight.</p> |  |
| <p>Handle the cuff carefully and avoid all types of stress through twisting or buckling in order not to damage the sensitive air-tight bubble.</p> |  |
| <p>Clean the device with a soft, dry cloth. Do not use gas, thinners or similar solvents. Spots on the cuff can be removed carefully with a damp cloth and soapsuds. Do not submerge the cuff in water!</p> |  |
| <p>Handle the tube carefully. Keep the hose away from sharp edges.</p> |  |

Do not drop the monitor or treat it roughly in any way.

Never open the monitor! Otherwise the manufacturer's calibration will be spoiled.

NOTE: According to international standards, your monitor should be checked for calibration every 2 years.

SYMBOL DESCRIPTION

| SYMBOL | REFERENT |
|--------|--|
| | Consult instructions for use |
| | Manufacturer |
| | Serial number |
| | Caution, consult accompanying documents |
| | Dispose of in accordance with the requirements of your country |
| | Type BF equipment |

| | |
|---|----------|
|  | Class II |
|  0413 | CE mark |

TECHNICAL SPECIFICATIONS

| | |
|----------------------|--|
| Measuring Method | Oscillometric |
| Indication | Digital LCD display |
| Measuring Range | Pressure: (30~280) mmHg Pulse: (40~199) Beats/min |
| Accuracy | Static pressure: ± 3 mmHg. Pulse: $\pm 5\%$. |
| Memory: | Last measurement memory |
| Power Supply | 4x1.5V batteries (LR6 or AA) |
| Operating Conditions | +5°C~+40°C, 15% RH~93% RH Atmospheric pressure: 70 kPa~106 kPa |
| Storage Conditions | -20°C~+55°C, 0% RH~93% RH Atmospheric pressure: 50 kPa~106 kPa |
| Dimensions | Approx: 62(D) X 135(H) X 90(W) mm |
| Weight | Approx: 380 g, excluding batteries |
| Classification | Type BF |
| Cuff size | 22-32 cm |
| AC Adapter | Input: 100 - 240 V, ~50/60 HZ, 0.2 A Output: 6 V $\overline{\overline{=}}$ 500 mA |

* The specifications and the outer design are subject to change for improvement reasons without previous notice.

WARRANTY

Your Automatic Blood Pressure Monitor Model M-100A is warranted for 5 years from date of purchase. Warranty for the cuff is 1 year from the date of purchase. The warranty does not apply to damage caused by improper handling, accidents, not following the operating instructions or self-maintained alterations made to the device.

The warranty is only valid upon presentation of the warranty card which was correctly filled in and sealed.

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For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

**SWISS
ENERGY
HERBS**

By Dr. FREI®

Black Garlic

with Vitamins B-Group

Heart&Brain

Vitality and Longevity

- ✓ Cholesterol level control
- ✓ Support in blood pressure decrease
- ✓ Aid for glucose level stabilisation
- ✓ Fight against infections
- ✓ Source of vital energy

**No odor
No gastric irritation**

Composition (in 2 capsules)

| | |
|---|--------------|
| Black garlic extract (Argentina origin) | 28% |
| Vitamin B1 | 1.1 mg/100%* |
| Vitamin B2 | 1.4 mg/100%* |

* Nutrient Reference Value


swissenergy-vitamins.com

Made in Switzerland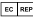

■ Shenzhen Pango Electronic Co., Ltd., No. 25 1st Industry Zone, Fenghuang Rd.,
Xikeng Village, Henggang Town, Longgang District, Shenzhen City, Guangdong Province,
China.  Lotus NL B.V., Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague,
Netherlands.

medpack-group.com
dr-frei.com

CE ISO
0413 13485