KZ

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home-test.ch

INSTRUCTION MANUAL HCG Urine Express Test (Test Strip) for Early Detection of Pregnancy

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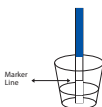
Home Test express test is intended for early detection of pregnancy.

How test works:

Home Test pregnancy test detects the presence of Human Chorionic Gonadotropin (HCG) hormone in the urine. The concentration of HCG in the urine of non-pregnant woman is normally 5.0 mIU/ml.

HCG peak levels from 100,000 to 200,000 mIU/ml are seen at the end of the first trimester. This pregnancy test detects HCG in urine at the concentration of 10mIU/ml or greater. This level can be detected within 7-10 days after conception.


Test procedure:

Test should be performed at room temperature. Remove the test strip from the sealed pouch, and perform the test immediately. Place the test strip vertically into the urine sample; make sure the arrows are pointing down. Do not immerse the test strip below MAX (maximum) line (marked by arrows on the test strip). Take the strip out of the urine sample after 5 seconds, and lay it flat on the clean, dry surface. Wait for colored bands to appear.

Test result can be observed not earlier than 5 minutes, but not later than 10 minutes.

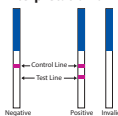Interpretation of results:

Negative:

If one colored line appears in the control region on the test strip and there is no line in the test region, then this indicates a negative result, no HCG has been detected in the urine. This means you are either not pregnant or you have tested too early. If you are not sure repeat the test in 48 hours.

Positive:

If two colored lines appear on the test strip, one in the test region and the other in the control region, then this means there is a strong possibility that you are pregnant – HCG has been detected in your urine. One line may be lighter than the other, they do not have to match and this is still a positive result.

Invalid:

If no lines appear in both test region and control region of the test strip, the test is invalid and you should repeat it using another test strip.

Attention:

1. It is not recommended to perform the test after alcohol consumption, because it can affect test result.
2. If you perform the test in the period just after conception, HCG level can be low, and the result will be negative. In this case repeat the test in 48 hours.
3. HCG level can be detected by the test during some days after labor, caesarean section, miscarriage, abortion.
4. Before making any decision as for medical measures, consult a doctor. Do not make any decision taking into account only this test results.

Storage conditions:

The test kit should be stored at room temperature (from 2°C to 30°C) in a sealed foil pouch to the date of expiration. Check the expiration date before use. The test kit should be kept away from direct sunlight, moisture, and heat. After usage throw the test strip in a waste bin.

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