AUTOMATIC BLOOD PRESSURE MONITOR

MODEL M-200A
USER'S MANUAL

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DEAR DR. FREI AUTOMATIC BLOOD PRESSURE MONITOR OWNER

Thank you for choosing Automatic Blood Pressure Monitor **Dr. Frei**® TM model M-200A. We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of Swiss Trademark **Dr. Frei**®.

Before starting to use this device please study the user's manual carefully. The user's manual offers all information you need to measure your blood pressure and pulse correctly. For all questions concerning the device please contact your local distributor or **Dr. Frei**® service centre in your country.

ATTENTION

This blood pressure monitor is designed to carry out self-control over blood pressure but NOT to make self-diagnosis of hypertension/hypotension. Please DO NOT diagnose by yourselves basing on the measurement results obtained with the blood pressure monitor. Please DO NOT execute self-treatment of high/low blood pressure and DO NOT change the methods prescribed without consulting your doctor.

Type BF applied part.

Read the instructions carefully before using this device.


TABLE OF CONTENTS

IMPORTANT INFORMATION ON BLOOD PRESSURE AND ITS MEASUREMENT	6
What Is Blood Pressure?	6
Which Values Are Normal?	7
ADVANTAGES OF AUTOMATIC BLOODPRESSURE MONITOR MODEL M-200A	8
Blood Pressure Rate Indicator	8
Irregular Heartbeat Detector	8
GETTING READY FOR MEASUREMENT	9
Safety Precautions	9
Components of Blood Pressure Monitor Kit	9
Description of the Blood Pressure Monitor	10
Inserting the Batteries	10
Setting the Time and Date	11
Selecting the User	11
Using an A/C Power Adapter	11
Cuff Connection	12
MEASUREMENT PROCEDURE	12
Fitting the Cuff	13
Measuring Procedure	13
Reading Measurement Result	14
Discontinuing a Measurement	14
MEMORY FUNCTION	14
Viewing the Stored Values	14
Delete Memories	15
ERROR MESSAGES/TROUBLESHOOTING	15
CARE AND MAINTENANCE	17
SYMBOL INFORMATION	18
TECHNICAL SPECIFICATIONS	18
WARRANTY	19

IMPORTANT INFORMATION ON BLOOD PRESSURE AND ITS MEASUREMENT


What Is Blood Pressure?

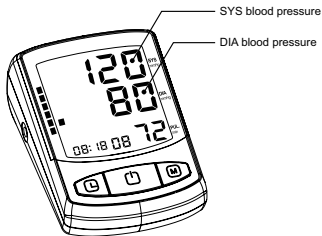Blood pressure is the pressure that your blood exerts on the vascular walls. Blood pressure is necessary to provide for constant blood flow inside the body. Thanks to it the cells get oxygen that provides for their normal functioning. The heart performs the function of a «pump», sending blood to the blood vessels. Each heart beat creates a certain level of the blood pressure.

There are 2 kinds of blood pressure: a systolic (upper) one, which corresponds to the heartbeat pushing blood into the arteriae; and a diastolic (lower) one, which means the blood pressure between two heartbeats. Blood pressure is subject to fluctuations during the day even in healthy people. The fluctuations are influenced by a number of factors - time of day, person's condition, physical or mental activity, environment, etc.

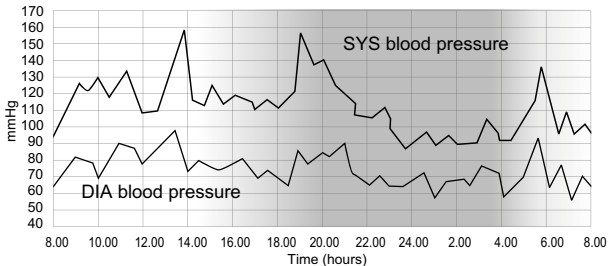Day-Night Fluctuations of the Blood Pressure

An increase of blood pressure increases the burden onto the heart, affects blood vessels making their walls thick and less elastic.

One of the features of the hypertension is its ability to remain unnoticed for the patient at its early stages. That's why the self-control of the blood pressure is so important. With the illness progressing, headaches and regular dizziness appear, the sight declines, the functioning of vitalses (encephalon, heart, kidneys, blood vessels) breaks down. Without special treatment the complications of hypertension might be kidney damages, breast-pang, paralytic stroke, aphasia, dementia, heart attack and stroke.

Which Values are Normal?

The world standard as for the norms of the blood pressure is the Classification* of the World Health Organization (WHO):

Category	Systolic Blood Pressure (mmHg)	Diastolic Blood Pressure (mmHg)
Blood pressure too low	< 100	< 60
Blood pressure optimum	100 - 119	60 - 79
Blood pressure normal	120 - 129	80 - 84
Blood pressure slightly high	130 - 139	85 - 89
Blood pressure too high	140 - 159	90 - 99
Blood pressure far too high	160 - 179	100 - 109
Blood pressure dangerously high	≥ 180	≥ 110

* printed with curtailments

- The diagnosis of hypertension requires from the patient to combine medical treatment prescribed by the doctor and mode of life correction.
- People with normal pressure and high normal pressure are recommended to carry out self-control of their tension in order to timely take measures to decrease the blood pressure level down to the optimal one without using any medications.
- For people more than 50 years old high level of systolic blood pressure (higher than 140 mmHg) is more crucial than diastolic pressure.
- Even with blood pressure being normal, people run the bigger risk of hypertension development with advancing age.

ATTENTION

If you have normal results of blood pressure measured under calm conditions but your results are excessively high when measured under the conditions of physical or mental exhaustion, this might be a sign of so called brittle (that is unstable) hypertension. If you suspect that, please consult your doctor.

When measured correctly, if diastolic blood pressure is more than 120 mmHg, it is necessary to call the doctor immediately.

ADVANTAGES OF AUTOMATIC BLOOD PRESSURE MONITOR MODEL M-200A

Blood Pressure Rate Indicator

Blood pressure rate indicator is located along left side of the display. The classification corresponds to 6 ranges described in the table of the section «Which values are normal?». After the measurement there appears the dotted line in the left part of the display opposite the zone to which the result of the measurement corresponds: green zone - optimal blood pressure, yellow - elevated, orange - too high, red - dangerously high. This function helps you to self-orient in the measurement results.

Irregular Heartbeat Detector

This technology indicates allorhythmic heartbeating. If the symbol of IHB  appears on the display that means that certain abnormality in heart beat frequency was detected during the measurement.

In this case, the result may deviate from your normal blood pressure - repeat the measurement. In most cases, this is no cause for concern. However, if the symbol  appears on a regular basis (e.g. several times a week with measurements taken daily) we advise you to tell your doctor. Please show your doctor the following explanation:

Information for the doctor on frequent appearance of the Irregular Heartbeat Detector.

This instrument is an oscillometric blood pressure monitor that also analyses pulse frequency during measurement.

The symbol  is displayed after the measurement, if pulse irregularities occur during measurement. If the symbol  appears more frequently (e.g. several times per week on measurements performed daily) we recommend the patient to seek medical advice. The instrument does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

GETTING READY FOR MEASUREMENT

Safety Precautions

1. Use this device **ONLY** for the intended use described in this manual.
2. Do **NOT** use accessories which are not specified by the manufacturer.
3. Do **NOT** use the device if it is not working properly or damaged.
4. Do **NOT** use the equipment where aerosol sprays are being used, or where oxygen is being administered.
5. Do **NOT** use under any circumstances on newborns.
6. This device does **NOT** serve as a cure for any symptoms or diseases. The data measured are for reference only. Always consult your doctor to have the results interpreted.
7. Keep the equipment and its flexible cord away from hot surfaces.
8. Do **NOT** apply the cuff to areas other than the place directed.
9. Proper maintenance and periodically calibration are essential to the longevity of your device. If you are concerned about your accuracy of measurement, please contact local customer service for help.
10. This system is intended for individuals age 18 or above.
11. Do not use it for diagnosis of hypertension or testing on newborns, babies, young children or persons who cannot express their consent.
12. If you have been diagnosed with arrhythmia or irregular heartbeat, atrial or ventricular premature beats or atrial fibrillation, measurements made with this instrument should only be evaluated after consultation with the doctor or your healthcare professionals.

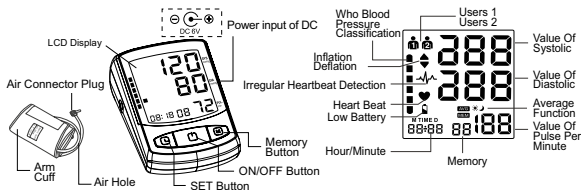
KEEP THIS MANUAL DURING THE WHOLE LIFETIME OF THE DEVICE

Components of Your Blood Pressure Monitor Kit

The blood pressure monitor kit includes Automatic Blood Pressure Monitor model M-200A, 22-42 cm cuff, 4 batteries (AA), user's manual, adapter, warranty card, kit box.

Description of the Blood Pressure Monitor

Measuring unit Model M-200A


Accessory:

* Please use the original adapter in accessory, otherwise it is impossible to achieve the claimed performance.

Inserting the Batteries

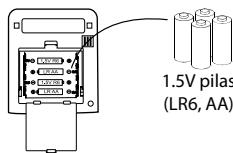1. Remove the battery cover.
2. Insert new batteries into the battery compartment as shown, taking care
3. that the polarities (+) and (-) are correct. Close the battery cover. Use only LR6, AA batteries.

CAUTION

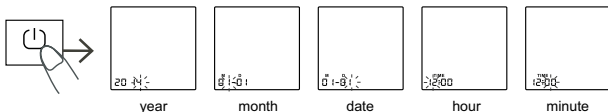
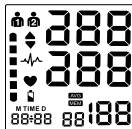
- Insert the batteries as shown in the battery compartment.
- When \square blinks in the display, replace all batteries with new ones. Do not mix old and new batteries. It may shorten the battery life, or cause the device to malfunction.
- \square does not appear when the batteries run out.
- Battery life varies with the ambient temperature and may be shorter at low temperatures.
- Remove the batteries if the device is not to be used for a long time.
- The batteries may leak and cause a malfunction.
- Use the specified batteries only. The batteries provided with the device are for testing monitor performance and may have a shorter life.

NOTE: To prolong the batteries life the device switches off automatically if no button is pressed for 1 minute. Otherwise you can switch it off by pressing the ON/OFF button.

Setting the Time and Date

This blood pressure monitor incorporates the function of time and date display. Time and date are recorded along with pressure and pulse.

1. Press the «  » button to switch on the monitor.
2. Press and hold «  » button until the year number displays and flashes on LCD to enter setting mode.
3. Press «M» to adjust the year. Confirm your selection by pressing «  » button.
4. Following the same steps to choose month, date, hour, minutes.

Select the User

This blood pressure monitor is designed to store 30 measurements for each of two users. Before taking a measurement, be certain that the correct user has been selected.

1. With the unit off, press «SET» button until the user icon in the upper left corner of the LCD screen flashes.
2. Press the «SET» button again to toggle between users.
3. Press the «ON/OFF» button to make your selection.

Using an A/C Power Adapter

It is possible to operate Automatic Blood Pressure Monitor model M-200A with an AC/DC adaptor.

1. Push the plug into the socket at the bottom part of the device. No power is taken from the batteries while the AC/DC adaptor is connected to the device.

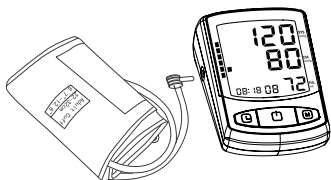
2. Plug the AC adaptor into a 220/110 V power socket. The device is ready for measurement.
3. Test that power is available by pressing the ON/OFF button.

NOTE:

- Please use the AC adaptor of Trademark **Dr. Frei®**. Any other adaptor being used, the service center is not considered responsible for the warranty servicing/repair of the monitor.
- No power is taken from the batteries while the AC/DC adaptor is connected to the device.
- Please consult service center if you have questions relating to the AC/DC adaptor.

Cuff Connection

Insert the cuff connector into the opening provided on the side of the device as shown in the picture.


MEASUREMENT PROCEDURE

ATTENTION

- Find time to relax by sitting in a quiet atmosphere for some time before measurement.
- Efforts by the patient to support the arm can increase the blood pressure. Make sure you are in a comfortable, relaxed position and do not flex the arm during measurement.
- Always measure on the same arm (normally left).
- Remove any garment that fits closely to your upper arm. Do not roll the sleeve since it can squeeze your hand and this can lead to false results.
- Use only clinically approved original cuff.
- If you want to follow the results of your blood pressure measurements, always perform measurements at the same time of day, since blood pressure changes during the course of the day.
- Measurements should be done after a 5 minute rest to ensure accuracy.

Fitting the Cuff

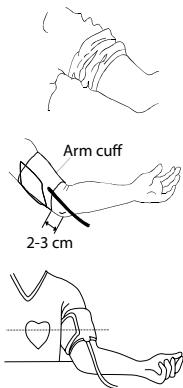
1. Plug the cuff connector to the cuff socket on the device left side.
2. Stretch your left (right) arm in front of you with your palm facing up. Slide and place the cuff onto your arm to let the air tube and artery mark region toward the lower arm. Wrap and tighten the cuff above your elbow. The artery mark on the edge of the cuff should be approx. 0.8" to 1.2" (2 cm to 3 cm) above your elbow. Align the tube over the main arteries on the inside.
3. Leave a little free space between the arm and the cuff: you should be able to fit two fingers between them. Remove all clothing covering or constricting the measurement arm.
4. Press the hook material firmly against the pile material. The top and bottom edges of the cuff should be tightened evenly around your upper arm.


Measuring Procedure

Always apply the pressure cuff before turning on the monitor.

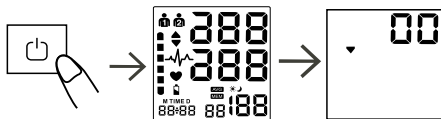1. Press ON/OFF button. All icons appear two seconds on DISPLAY, then switch to measurement, and display "0" or last measurement record.

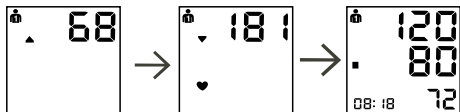2. Pump begins to inflate the cuff. The heart symbol «♥» is shown on the display. When complete, the results will be displayed.



Reading Measurement Results

When the measurement has been concluded, the measured blood pressure, pulse rate and color bars classified by WHO (World Health Organization) are now displayed. The appearance of this symbol  signifies that an irregular heartbeat was detected. This indicator is only a caution.

NOTE: We recommend contacting your physician if you see this indicator frequently.

It is important that you be relaxed, remain still and do not talk during measurements.

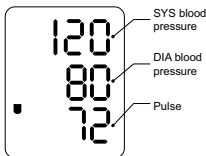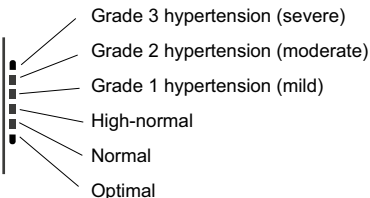On the graphic indicator the element showing level of arterial pressure on a scale of World Health Organization is displayed.

Discontinuing a Measurement

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels unwell), the ON/OFF button be pressed at any time. The device then immediately lowers the cuff pressure automatically.

MEMORY FUNCTION

At the end of a measurement, this monitor automatically stores each result with date and time. This unit stores 30 memories for each of 2 users.

Viewing the Stored Values

With the unit off, press «M» button. The display first shows **AVG**, then shows the average of the last three measurements within 10 minutes. Please note: During the last 10 minutes of the unit, if there is no memory stored, it shows

«0»; if only the last memory is stored, it shows the last memory; if there are 2 memories stored, it shows the average of the 2 memories. Besides, measurements for each user are averaged and stored separately. Be certain that you are viewing the measurements for the correct user.

Pressing the MEMORY button again displays the previous value. To view a particular stored memory, press the MEMORY button to scroll to that stored reading. If an irregular heartbeat is detected, the symbol \sim is displayed including the time and the result of the measurement.

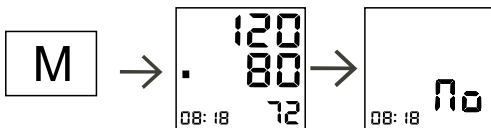
Further information

Measurements should not occur soon after each other, since otherwise, the results will be falsified. Wait therefore for several minutes in a relaxed position, sitting or lying, before you repeat a measurement.

Delete Memories

Before you delete all memory readings stored in, make sure you won't need refer to them at a later date. Keeping a written record is prudent and may provide additional information for your doctor's visit.

Delete memory (all the memory will be deleted) hold «M» for 5 seconds, display «No» means all the memories have been deleted.

ERROR MESSAGES/TROUBLESHOOTING

If you have trouble in using the unit please check the following points first.

ERROR DISPLAY	POSSIBLE CAUSE	HOW TO CORRECT
Nothing is displayed when you push the ON/OFF button or  battery icon flash	No battery installation	Insert batteries
	Battery worn out	Replace with new batteries
	The polarities of batteries placed wrongly	Insert battery in the correct polarities


E1: Can't normally increase pressure	Check your cuff for air leakage	Replace cuff with new one
E3: Inflate pressure too high		Remeasure or send back to dealer to recalibrate
E2 E4: Hand shaking while measurement	Hand or body shaking while measurement	Keep static position and measure again
Battery icon on	Battery low power	Replace battery and measure again
The systolic pressure value or diastolic pressure value too high	The cuff was held lower than your heart	Keep correct position and measure again
	The cuff was not attached properly	
	You moved your body or spoke during measurement	
The systolic pressure value or diastolic pressure value too low	The cuff was held higher than your heart	
	You moved your body or spoke during measurement	

ATTENTION

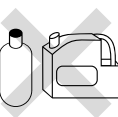Should any technical malfunctions arise in the blood pressure monitor, please contact the the authorized service centre of **Dr. Frei®**. **Never attempt to repair the instrument yourself!** Any unauthorized opening of the instrument invalidates all warranty claims.

NOTE: Blood pressure is subject to fluctuations even in healthy people. Please remember that comparable blood pressure measurements always require the same time and the same conditions! These are normally quiet conditions. If you follow the correct procedures described above and still get the fluctuations of blood pressure of more than 15 mmHg and/or you repeatedly hear irregular pulse tones please consult your doctor.

CARE AND MAINTENANCE

<p>Do not expose the device to either extreme temperatures, humidity, dust or direct sunlight.</p>	
<p>Handle the cuff carefully and avoid all types of stress through twisting or buckling in order not to damage the sensitive air-tight bubble.</p>	
<p>Clean the device with a soft, dry cloth. Do not use gas, thinners or similar solvents. Spots on the cuff can be removed carefully with a damp cloth and soapsuds. Do not submerge the cuff in water!</p>	
<p>Handle the tube carefully. Keep the hose away from sharp edges.</p>	
<p>Do not drop the monitor or treat it roughly in any way.</p>	
<p>Never open the monitor! Otherwise the manufacturer's calibration will be spoiled.</p>	

NOTE: According to international standards, your monitor should be checked for calibration every 2 years.

SYMBOL INFORMATION

SYMBOL	REFERENT
	Consult instructions for use
	Manufacturer
	Serial number
	Caution, consult accompanying documents
	Dispose of in accordance with the requirements of your country
	Type BF equipment
	Class II
	CE mark

TECHNICAL SPECIFICATIONS

Measuring Method	Oscillometric
Indication	Digital LCD display
Measuring Range:	Pressure: (30~280) mmHg Pulse: (40~199) Beat/min
Accuracy:	Static pressure: ± 3 mmHg Pulse: $\pm 5\%$

Memory:	2 x 30 Memories
Power supply:	4x1.5 V Batteries (LR6 or AA)
Operating condition:	+5°C~+40°C, 15% RH~93% RH Atmospheric pressure: 70 kPa~106 kPa
Storage condition:	-20°C~+55°C, 0% RH~93% RH Atmospheric pressure: 50 kPa~106 kPa
Dimensions:	Approx: 90 (W) X 135 (H) X 62 (D) mm
Weight:	Approx: 380 g, excluding batteries
Classification	Type BF
Cuff size	22-42 cm
AC Adapter	Input: 100-240 V, ~50/60 HZ, 0.2 A Output: 6 V $\overline{\text{---}}$ 500 mA

* The specifications and the outer design are subject to change for improvement reasons without previous notice.

WARRANTY

Your Automatic Blood Pressure Monitor Model M-200A is warranted for 5 years from date of purchase. Warranty for the cuff is 1 year from the date of purchase. The warranty does not apply to damage caused by improper handling, accidents, not following the operating instructions or self-maintained alterations made to the device.

The warranty is only valid upon presentation of the warranty card which was correctly filled in and sealed.

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For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Shenzhen Pango Electronic Co., Ltd., No. 25 1st Industry Zone, Fenghuang Rd.,
Xikeng Village, Henggang Town, Longgang District, Shenzhen City, Guangdong Province,
China.   Lotus NL B.V., Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague,
Netherlands.

global-swiss.ch
dr-frei.com

CE **ISO**
0413 13485