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ART. 8503

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## WRIST WRAP ELASTIC

## Indications for use:

Can be used for trauma, sprain, arthrosis and arthritis, while rehabilitation after plaster cast. Supports also used for prevention of common dislocations and traumas while physical exercises.


## Application:

- Provides soft fixation and compression for wrist joint.
- Breathable fabrics stretches in four directions for more comfortable usage.
- The anatomical shape is convenient for daily use.
- Fits for right and left arm.

## Contraindications:


Inflammatory skin disease in the area of brace application, individual intolerance to the brace materials.

Dimensions	Sizes				Max deviation
	S	M	L	XL	
Wrist circumference, cm	14-16	17-19	20-22	23-25	± 1,0 cm

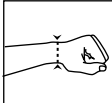
## How to get it on:

for left hand

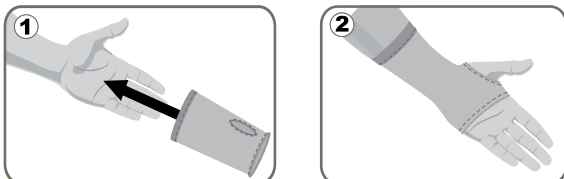for right hand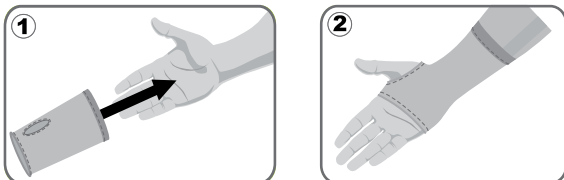

## Maintenance:

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

## Precautions:

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

**Contents:** cotton - 75%, latex - 17%, polyether - 8%.

**Storage conditions:** store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

**Manufacturing date:** see on packaging.

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MedPack Swiss GmbH ist ein Mitglied der MedPack Swiss Group, Schweiz.