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# INSTRUCTION MANUAL HCG Urine Express Test (Midstream Test) for Early Detection of Pregnancy

Home Test express test is intended for early detection of pregnancy.

## How test works:

Home Test pregnancy Midstream test detects the presence of Human Chorionic Gonadotropin (HCG) hormone in the urine. The concentration of HCG in the urine of non-pregnant woman is normally 5.0mIU/ml. HCG peak levels from 100,000 to 200,000mIU/ml are seen at the end of the first trimester. This Midstream test detects HCG in urine at the concentration of 10mIU/ml or greater. This level can be detected within 7-10 days after conception.

## Test procedure:

Midstream test is easy to use. There is no need to collect urine. Test should be performed at room temperature. Open the sealed pouch, take out the test device and remove the protective cap. Place the absorbent tip marked arrow under urine stream for 5-10 seconds to be thoroughly wet. Re-cap the device and wait for color bands to appear. Results can be observed not earlier than 5 minutes but not later than 10 minutes.

## Interpretation of results

## Negative:

If one colored line appears in the control region on the Midstream test and there is no line in the test region, then this indicates a negative result, no HCG has been detected in the urine. This means you are either not pregnant or you have tested too early. If you are not sure repeat the test in 48 hours.

## Positive:

If two colored lines appear on the Midstream test, one in the test region and the other in the control region, then this means there is a strong possibility that you are pregnant – HCG has been detected in your urine. One line may be lighter than the other, they do not have to match and this is still a positive result.

## Invalid:

If no lines appear in both test region and control region of the Midstream test, the test is invalid and you should repeat it using another Midstream test.

## Attention:

1. It is not recommended to perform the test after alcohol consumption, because it can affect test result.
2. If you perform the test in the period just after conception, HCG level can be low, and the result will be negative. In this case repeat the test in 48 hours.
3. HCG level can be detected by the test during some days after labor, caesarean section, miscarriage, abortion.
4. Before making any decision as for medical measures, consult a doctor. Do not make any decision taking into account only this test results.

## Storage conditions:

The test kit should be stored at room temperature (from 2°C to 30°C) in a sealed foil pouch to the date of expiration. Check the expiration date before use. The test kit should be kept away from direct sunlight, moisture, and heat. After usage throw the midstream test in a waste bin.

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