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EC REP MedPack Swiss GmbH, Switzerland, Tramstrasse 16, 9442 Berneck,

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## MATERNITY SUPPORT ELASTIC

**Indications for use:**

Support is used during pregnancy to reduce pains at lower back and hips. Provides soft support to the abdomen and secures the best conditions for the correct fetus position. Due to 2 removable rigid strips it relieves stress from the lower back and supports the abdomen during pregnancy. Breathable elastic material provides comfort at any season.

- Reduces pains at lower back and hips
- Prevents striae cutis distensae
- Helps to maintain correct posture and active lifestyle
- Invisible under clothing

**Application:**

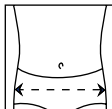
Before use please consult with your doctor.

Required size is chosen based on the hip circumference shown in the chart above. It's recommended to wear all the day since morning. In the antepartum period wear the support while you are lying on the back with hips elevation. Wear directly on underwear. Install wide part on the waist, and fix under the stomach with Velcro fastener. Before going to bed the support must be put off. This model can be used after the delivery for abdominal muscles tone renewal. At that it is necessary to extract plastic ribs from the wide part of the support. Individual treatment and terms are determined by the physician.

**Contraindications:**

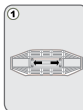
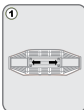
Skin local diseases, individual intolerance of the raw material components.

Dimensions	Sizes						Max deviation
	S	M	L	XL	XXL	XXXL	
Hips circumference, cm	89-97	98-106	107-115	116-124	125-134	135-144	± 1,0 cm

**How to get it on:**

schemes of bandage use before delivery

schemes of bandage use after delivery

**Maintenance:**

Wash with mild detergents at temperature no higher +35°C. Don't rub or wring out during washing and rinsing. Dry up away from sources of heat. Don't iron. Don't bleach. (Don't use chlorine-containing bleaching detergents). Don't use dry-cleaning.

**Precautions:**

Use the product for its intended purpose only. The schedule or time of wearing can be changed upon your doctor's advice only. It is not recommended to wear the product within 30 minutes after application of local warming ointments.

**Contents:** polyether - 47%, latex- 41%, nylon - 12%, plastic splints

**Storage conditions:** store in dry place at temperature from +5°C to +40°C. Keep away from direct sun rays.

**Shelf life:** 36 months from manufacturing date observing storage conditions, exploitation and care of this product. After the termination of the storage period manufacturer does not guarantee sufficient elasticity of the product.

**Manufacturing date:** see on packaging.

**Size:** see on packaging

MedPack Swiss GmbH, Switzerland, Tramstrasse 16, 9442 Berneck,  
phone: +41 71 747 11 95, fax: +41 71 747 11 91, e-mail: info@medpack-group.com,  
www.medpack-group.com.

Technomedica LLC., 24-A Berezinskaya str, Dnepropetrovsk, Ukraine 49130,  
phone/fax: +38 056 378 99 56, e-mail: info@med-textile.com, www.medtextile-swiss.com.